


mediven plus thigh length

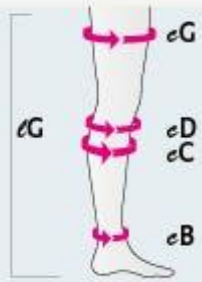

l = length measurement
e = circumference

All measurements should be taken on the bare leg in centimetres.
Ensure that the tape measure is snug but not too tight.
Measurements should be taken when the leg is rested and not swollen.

Please consider that selecting the right size is very important.

circumference in cm		I	II	III	IV	V	VI	VII
thigh (no topband)	(eG)	43 – 57	45 – 62	49 – 67	53 – 72	56 – 77	60 – 81	64 – 85
thigh (silicone basic)	(eG)	43 – 48	45 – 52	49 – 56	53 – 60	56 – 64	60 – 68	64 – 72
thigh (silicone x-wide)	(eG)	49 – 57	53 – 62	57 – 67	61 – 72	65 – 77	69 – 81	73 – 85
below knee	(eD)	27 – 33	29 – 36	32 – 39	34 – 42	36 – 45	38 – 48	40 – 50
calf	(eC)	28 – 34	30 – 37	33 – 40	35 – 43	37 – 46	39 – 49	41 – 51
above ankle	(eB)	18 – 20	20 – 22	22 – 24	24 – 26	26 – 28	28 – 30	30 – 32

length in cm

petite	(lG)	62 – 71
regular	(lG)	72 – 83

Suitable for these shoe sizes:
up to UK shoe size 9